

# PEEPIYA

BEDWETTING TREATMENT SOLUTION

## User Manual



## Charging



- When the battery power is low, the indicator will show red light, and it will flash at a frequency of twice in one second.
- When the battery runs out of power, this alarm will power off automatically and won't work.
- Plug the attached standard USB port to specified charger, and insert the Micro USB part to the body interface for charging. The indicator shows red light on.
- After fully charged, the indicator will turn to blue light.

### Notice

This alarm can be charging normally when it is in any one of 3 modes on the toggle switch; For initial use, please run the original battery power out first, then recharge it fully to use again so as to extend its lifetime; Please do not use this alarm when charging.

## Cleaning And Maintenance

After each use, please clean the sensor clip with fresh water in time, and wipe the water stains away from the surface or dry it with air blower, electric fan etc.

Please do not soak the alarm body or inductive clip into water, otherwise, the internal components will be damaged.

Please regularly check the built-in battery to ensure the alarm functions can work properly.

If you do not use it for a long time, we recommend to charge this alarm once a month or every two months to extend the battery life.

PEEPIYA Bedwetting Treatment Solution is exclusive for user, please do not hand it over to others en route.


## Customer Service

Qutek guarantees to provide one-year warranty service for your bedwetting alarm purchased from our store.

The warranty coverage excludes: Malicious damage, incorrect use and excessive wear.

If the alarm is not directly purchased from Qutek direct-sale store, but proof of purchase can be provided, customers can still enjoy our after-sales service.

## Instruction ①

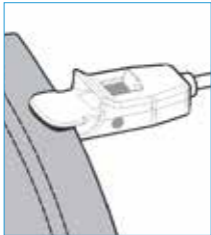

PEEPIYA Bedwetting Treatment Solution applied to: Toddler (infant, child), youngsters, enuresis / nocturia patients, incontinent patients, physically inconvenient people etc.

Recommended that you should proceed the product test before putting this alarm into use to ensure this alarm can work properly.

### Test method

When the switch is toggled to the mode of "Vibration/Sound and Vibration", please take one piece of damp cloth or paper to be clipped into the middle of inductive clip, if the alarm is vibrating or sounding and vibrating synchronously, it can be regarded as normal working status.

## Instruction ②

The alarm volume is confirmed and proved by massive testing and debugging. It can be applied to most of people.

It needs some time for initial users to adapt the alarm sound and vibration. After continuous use, they will get used to it. This alarm cannot awake users instantly in the early days, if users still wet the beds, please do not blame on them. What you can do is to ask their reaction and encourage them for emphasizing the importance of timely potty.

When the users gradually adapt to this alarm and awaked for timely potty and do not wet their bed sheet, please grant enough praise and encouragement.

Especially when this alarm is used in night time, it's recommended that you should install and turn on a night lamp for users to wake up easily and go to bathroom safely. If users are too young to act themselves, or it is inconvenient for them, family members should assist them to pee or change dry and clean clothes.

In the process of using, it does not need to limit excessive water intake.

This alarm will be very timely to awake deep sleepers and give them psychological hint when use it for long terms. It will help them to develop a good habit of peeing after alarm. At the same time, this alarm is helpful for toddlers to proceed potty training.

Additionally, using the progress card customized by Qutek will help to pay more attention to the improvement of the users.

## Operation Modes




Operation Modes: OFF/Vibration/Sound and Vibration

Working Status of 3 in 1 Toggle Switch on the alarm:

- It will be in the status of "OFF" when the switch is toggled in the middle part, and this alarm does not work.
- When the switch is toggled to the mode of "vibration", press one end of inductive clip, the metal parts will be touched, the alarm will vibrate and the indicator will become flickering blue light.
- As same as step two, while it is switched to the mode of "Sound and Vibration", the alarm will be sounding and motor vibrating accompanied with the flickering blue light.

## Operation Instruction

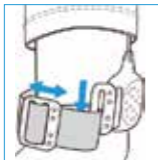

Step One: Open the elastic band inserted with hook, adjust the armband to the suitable circumference through the slide buckle, and then tie it on your child's arm.



Step Two: The clip of this alarm should be in the open mode, and fix the initial wetted part of your child's underwear on the middle of the clip.




Step Three: Push the toggle switch forward to the bottom for fixing firmly.

- Toggle the switch to the corresponding mode in conformity with current circumstances and conditions according to your personal requirements.
- In the process of usage, if your child is urinating, the fixed position of the clip will be wetting, subsequently the alarm will be warning. User should turn off the alarm and take down the clip to bathroom for urinating entirely and changing dry and clean clothes.
- Please clean the alarm sensor. If you need to use it again, please re-fix the clip well and open the alarm.

### Attention

We recommend that the users should wear cotton underpants; Depending on the actual conditions, adjust the clip to the fixed position, Fixing the clip on the initial wetted part of your child's underpants, the alarm effect will be better.

## Specification

Fuselage Diameter	5cm	
Signal Transmission Line Length	95cm	
Alarm Volume	50-82DB	
Input Voltage	5.0V	
Maximum Operating Current	150MA	
Built-in Lithium Battery Capacity	500mAh	
Built-in Lithium Battery Voltage	3.7V	
Battery Charging Duration	Approx . 2 hours	

## Structure

